


VITA shade taking

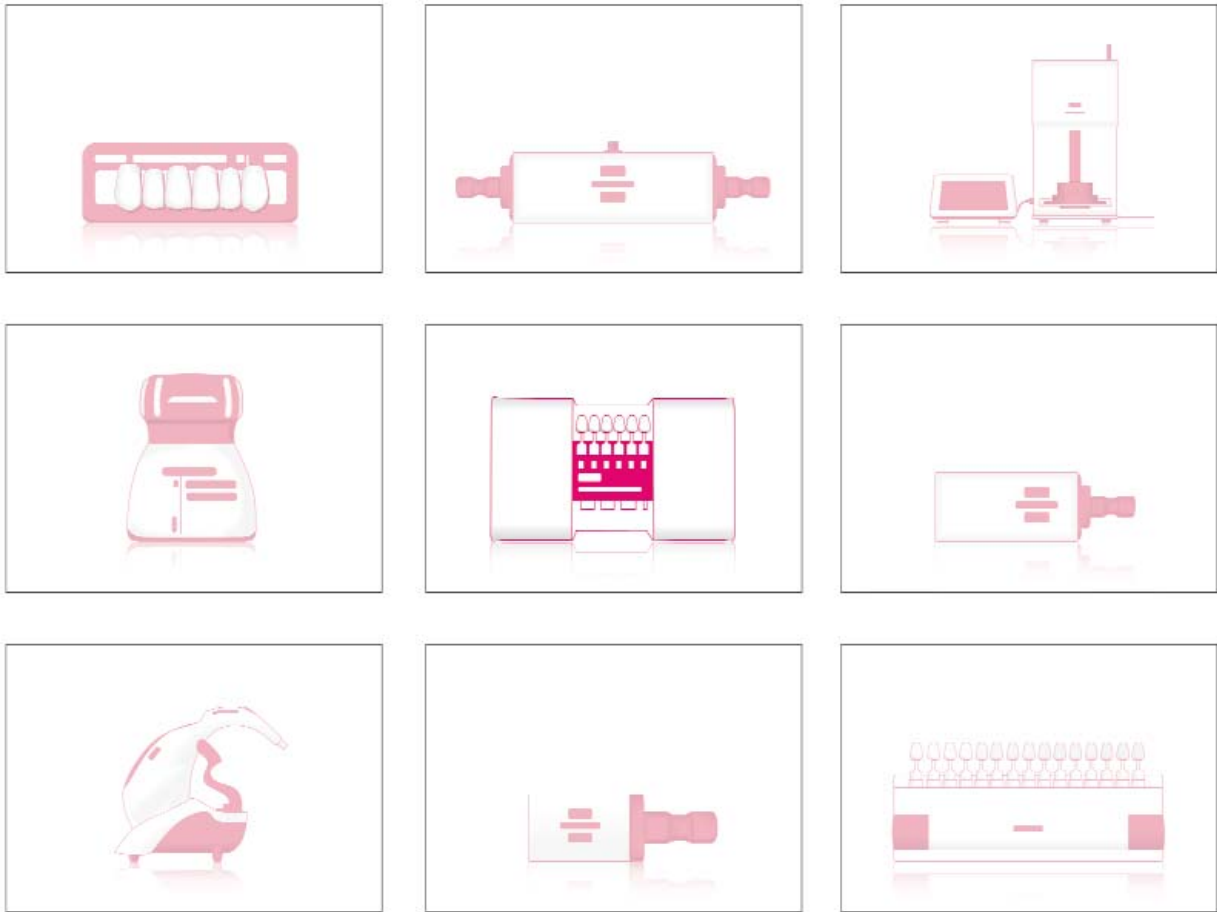

VITA shade, VITA made.

VITA

VITA Linearguide 3D-MASTER

The new simplicity in shade taking



Product description

With the VITA Linearguide 3D-MASTER you can determine the correct tooth shade swiftly and accurately. The modern design and systematic structure of the VITA Linearguide enable the appropriate 3D-MASTER shade to be found quickly.

The VITA Linearguide 3D-MASTER facilitates the changeover from traditional shade-taking systems to the precise shade determination and shade communication in the VITA SYSTEM 3D-MASTER.

The VITA Linearguide 3D-MASTER contains the same 29 shade samples as the VITA Toothguide 3D-MASTER.

Features/Benefits

- Classical, linear handling
- Self-explanatory, simple use
- Reliable, fast and precise shade determination for satisfied customers
- Modern design as an expression of perfect esthetics and a clear structure

VITA Linearguide products

VITA SYSTEM 3D-MASTER® shades

Description	Content	Prod. no.
VITA Linearguide 3D-MASTER	1 Box	B363

Printed materials

Title	Description	Order no.
VITA Linearguide 3D-MASTER	Instructions	920-02565
VITA Linearguide 3D-MASTER	information sheet	1532

VITA Toothguide 3D-MASTER[®] with BLEACHED SHADE GUIDE



Product description

VITA Toothguide 3D-MASTER[®]

With the VITA Toothguide 3D-MASTER, all natural tooth shades can be determined accurately and systematically.

Features/Benefits

- Determination of the tooth shade in 3 steps
- Unmistakable determination of the tooth shade
- Possibilities of comparison: Yes/No - Better/Worse decision
- Tooth shades precisely defined by science (miscible)
- Can be learned

Product description

BLEACHED SHADE GUIDE

The VITA BLEACHED SHADE GUIDE enables the shades of bleached teeth to be determined accurately and systematically using the same principle as the VITA Toothguide.

VITA Toothguide products

VITA SYSTEM 3D-MASTER® shades

Description	Content	Prod. no.
VITA Toothguide 3D-MASTER	1 pc	B360
VITA BLEACHED SHADE GUIDE, Shade Group 0	1 pc	B2600MS

Printed materials

Title	Description	Order no.
VITA Toothguide 3D-MASTER	Instructions	961
VITA Toothguide 3D-MASTER	Color communication form	914
VITA Toothguide 3D-MASTER with Bleached Shade Guide	information sheet	1419

VITA Bleachedguide 3D-MASTER®



Product description

The VITA Bleachedguide 3D-MASTER is a tool for the dentist for the planning and control of tooth lightening procedures.

The Composition follows the principles of the VITA SYSTEM 3D-MASTER based on the 3 shade parameters lightness, chroma and hue. In addition to the lightness and chroma levels, however, additional intermediate levels with supplementary shade samples have here been added.

Features/Benefits

- Tooth lightening can be discussed with the patient in realistic terms.
- Uniform color distances which can be clearly distinguished by the human eye
- Intermediate steps are available as shade samples (lightness, chroma)
- Dentist and patient can discuss the desired result on the basis of the shade samples and later check the result obtained.
- Prosthetic restorations can – if necessary – be adapted to the VITA SYSTEM 3D-MASTER.
- At a recall visit it can be checked whether it is necessary to repeat the tooth lightening procedure.

VITA Bleachedguide products

VITA SYSTEM 3D-MASTER® shades

Description	Content	Prod. no.
VITA Bleachedguide 3D-MASTER	1 pc	B361

Printed materials

Title	Description	Order no.
VITA Bleachedguide 3D-MASTER	Instructions	920-26266
VITA Bleachedguide 3D-MASTER	information sheet	1432
VITA Bleachedguide 3D-MASTER	Cardboard (paper) version	1486

VITA classical A1-D4 shade guide



Product description

The VITA classical A1 - D4 shade guide serves to accurately determine tooth shade.

The arrangement of the shades in the VITA classical family of shades is as follows:

A1 - A4 (reddish-brownish)

B1 - B4 (reddish-yellowish)

C1 - C4 (greyish shades)

D2 - D4 (reddish-grey)

Tips for shade-taking

- If possible, determine the shade under daylight conditions or standardised daylight lamps, and not under normal indoor lighting conditions.
- The shade-taking environment should preferably be in muted colours. Patients should be asked to remove lipstick or any cosmetics which could have an influence on shade-taking. Clothing in pronounced colours should be covered with a grey overall.
- Make your choice swiftly; always accept your first decision, since the eyes begin to tire after approx. 5 – 7 seconds.

VITA classical A1-D4 shade guide products

Description	Content	Prod. no.
VITA classical A1-D4 shade guide	1 pc	B0271

VITA classical A1-D4 shade guide with VITA Bleached Shades

The original is being expanded – Bleached Shades now available for VITA classical



Product description

VITA Bleached Shades offer the dentist a practical tool for the reliable determination and maintenance of a whitening regimen, and are now available for the VITA classical shade guide. With the additional VITA Bleached Shades, the tooth colours of bleached teeth are also displayed in the VITA classical shade guide.

The VITA Bleached Shades consist of the bleached shades 0M1, 0M2 and 0M3. These are taken from the VITA SYSTEM 3D-MASTER, and are also available as an add-on to the VITA classical A1 - D4 shade guide. The new VITA Bleached shades come with a holder that clips onto the VITA classical shade guide, offering a linear extension to the existing shade tabs.

The VITA Bleached Shades add-on is also available as attachment for VITA classical A1-D4 and VITAPAN classical.

Dentists preferring to remain with the VITA 3D-Master shade system, which offers more precise and reproducible shade measurement, will continue to benefit from the VITA Bleachedguide 3D-Master, which is designed specifically for tracking a bleaching regimen.

Tips for shade-taking

- If possible, determine the shade under daylight conditions or standardised daylight lamps, and not under normal indoor lighting conditions.
- The shade-taking environment should preferably be in muted colours. Patients should be asked to remove lipstick or any cosmetics which could have an influence on shade-taking. Clothing in pronounced colours should be covered with a grey overall.
- Make your choice swiftly; always accept your first decision, since the eyes begin to tire after approx. 5 – 7 seconds.

VITA classical A1-D4 shade guide with Bleached Shades products

Description	Content	Prod. no.
VITA classical A1-D4 shade guide with VITA Bleached Shades	1 pc	B027IV1
VITA Bleached Shades (extension set)	2 adapter, 3 shade samples	B027BS

VITA Easyshade® Advance

Evolution continues



Product description

VITA Easyshade Advance impresses with a new software, which contributes to increasing user-friendliness and performance of the instrument. Daily routines in the practice/laboratory can be completed since faster and more accurate results are achieved during shade taking.

The optimized instrument enables reliable and highly accurate shade taking of natural teeth and restorations completely independent of lighting conditions and user. Compared to traditional, visual shade determination, precisely defined measurement data eliminate errors when taking, communicating or interpreting the shade and guarantee higher economic efficiency of your practice/laboratory.

The new VITABLOCS mode of VITA Easyshade Advance enables the accurate determination of the VITABLOCS shade based on digital shade measurement so that tooth shade measurement has now also entered the CAD/CM era.

VITA Easyshade Advance is the next step towards perfection of tooth shade measurement technology to achieve fast, precise and objective measuring of shades.

Features/Benefits

- Reliable shade measurement within a few seconds
- Perfect determination in VITA classical A1-D4, 3D-Master and VITABLOCS shades
- Independent of ambient lighting
- Easy to use with maximum freedom of movement
- Prevention of infection guaranteed
- Reliable communication and documentation of shades
- Quality control thanks to the possibility to verify restorations
- Extended durability thanks to modern LED technology
- Stores up to 30 shades

VITA Easyshade[®] Advance kit

VITA Easyshade Advance

Prod.no. DEASYAS

Quantity	Content	Material
1	pc	Easyshade Advance Base Unit
1	pc	Easyshade Advance Handpiece
40	pcs	Infection Control Shields
1	pc	CD-Rom (ShadeAssist Software)
1	pc	Easyshade Advance Packing
1	pc	mains power supply
1	pc	USB cable
1	pc	Operating Manual
1	pc	Brief instructions for use



Printed materials

Title	Description	Order no.
VITA Easyshade Advance	Instructions for use	1506/1
VITA Easyshade Advance	Brief instructions for use	1559M/1
VITA Easyshade Advance	Brochure	1533

VITA Easyshade[®] Advance products

Description	Content	Prod. no.
Infection control shield	4 x 40 pcs	D39003
Transport case for Easyshade Advance	1 pc	DEASYASK
VITA ShadeAssist Software	1 pc CD-Rom	D46900

VITA ShadeAssist

Tooth colour determination software



Product description

The proven shade communication software was considerably improved again and numerous new features were added. Among others, the user has the possibility to send patient diagnoses and reports directly via email and to import them into their electronic patient records. Moreover, Easyshade measurements can now be imported and positioned in photos and additional notes and drawings can be attached. At the click of a button, translucent areas of the tooth are shown more clearly.

Product description

Thanks to a special analysis function to further facilitate the communication between dentists and laboratories. Processing patient data was also advanced and supports practices or laboratories in preparing clear documentation and communication of their patient data and hence also contributes to quality management. The new modified ShadeAssist software also communicates with the VITA Easyshade Compact. The software will be enclosed with all VITA Easyshade Advance systems from July 2011.

VITA ShadeAssist products

Description	Content	Prod. no.
VITA ShadeAssist	1 CD-Rom	D46800-04